



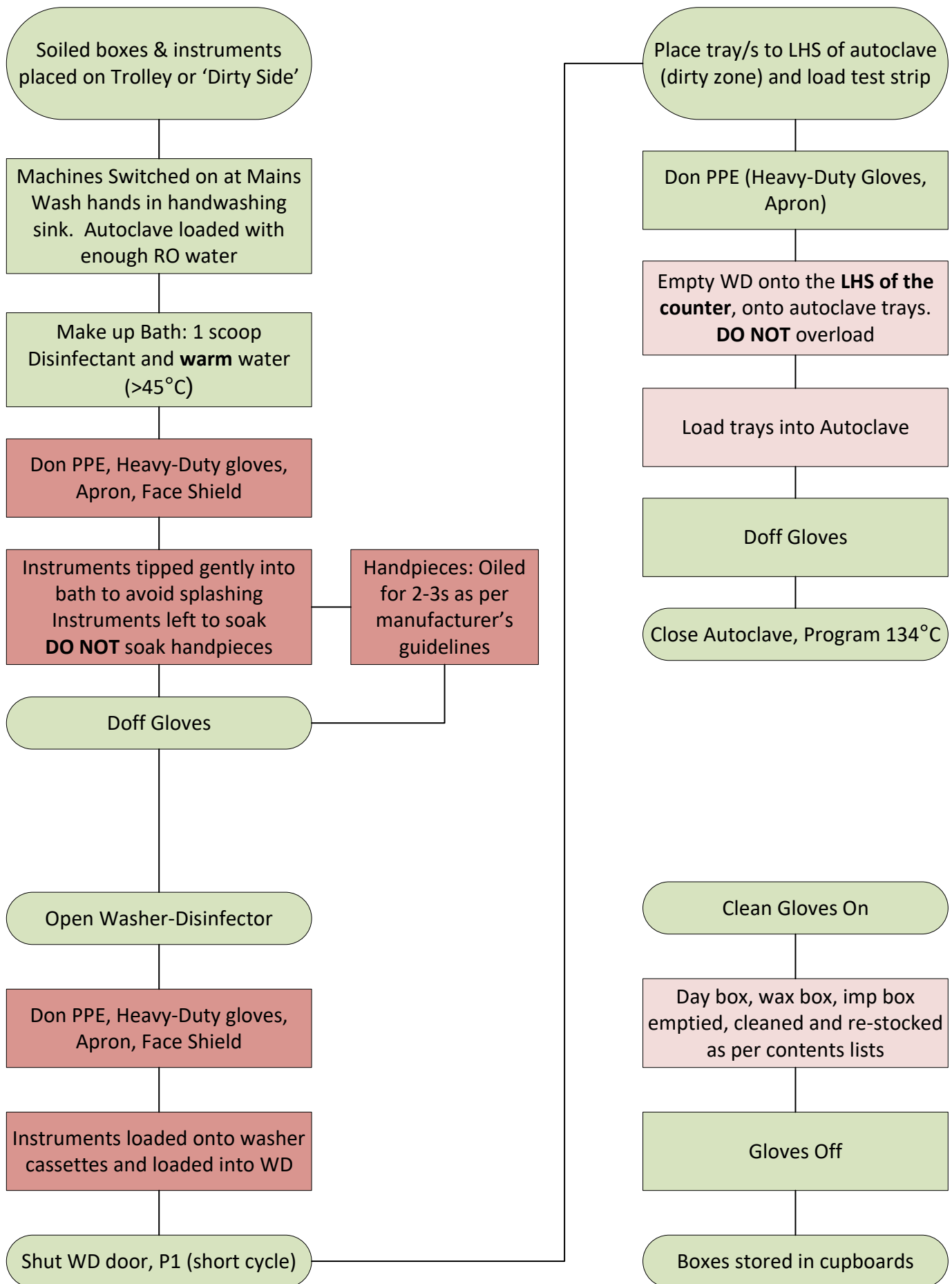

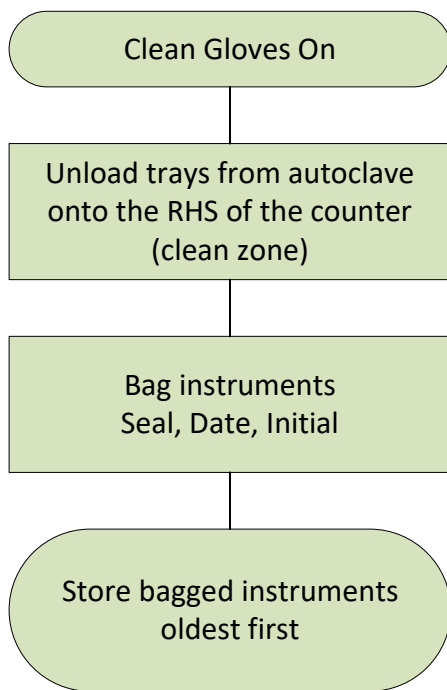

Daily:

- Log sheets for WD and Autoclave. Please complete even if not used
- Autoclave drained, door open and lid off
- Machines and surfaces cleaned
- Head Lamp on charge
- In the warmer months (>20°C ambient temperature), LA and glucagon to be taken out of car daily and kept in refrigerator. Reminder sign placed on driver's seat, or magnetic notice board. To be placed in mobile refrigerator in car for the day, along with ice pack

Weekly:

- Protein tests for autoclave
- Valisafe metal strip WD test
- Date and results recorded
- WD Descaled and recorded
- Light cure light intensity checked and recorded, light cure charged and 'On Charge' sign placed on driver's seat or on notice board